



## Control panel

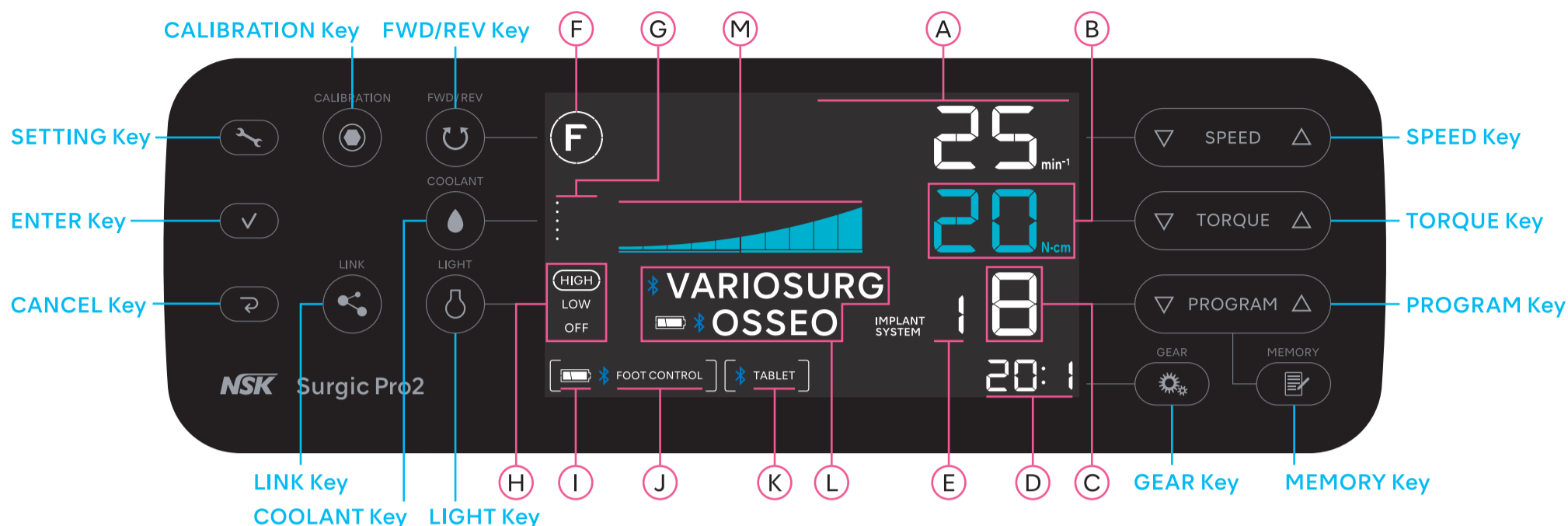


## Foot control

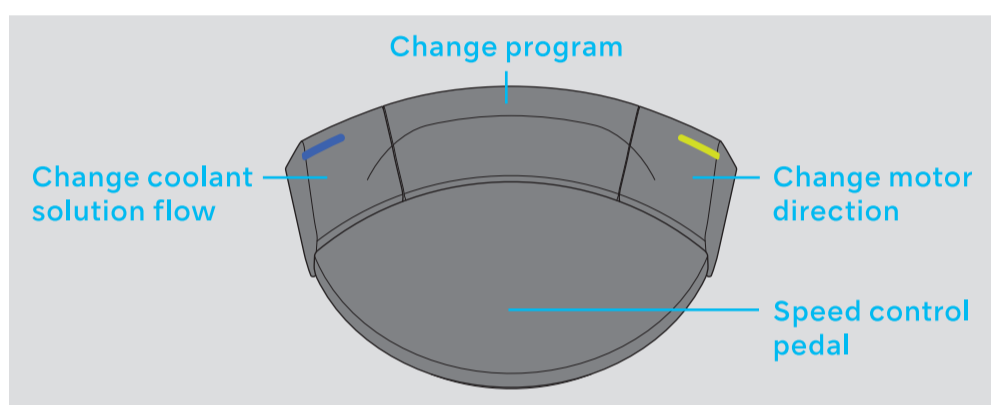


A	Rotation speed	I	Battery level of the foot control
B	Torque	J	Status of connection between the foot control
C	Program No.	K	Status of connection between an external terminal and the control unit
D	Gear ratio	L	Status of connection between VarioSurg 3 and Osseo 100+
E	Implant system No.	M	Current operating torque level
F	Rotation direction		
G	Coolant solution flow level		
H	Light level		

## Preparation for Use

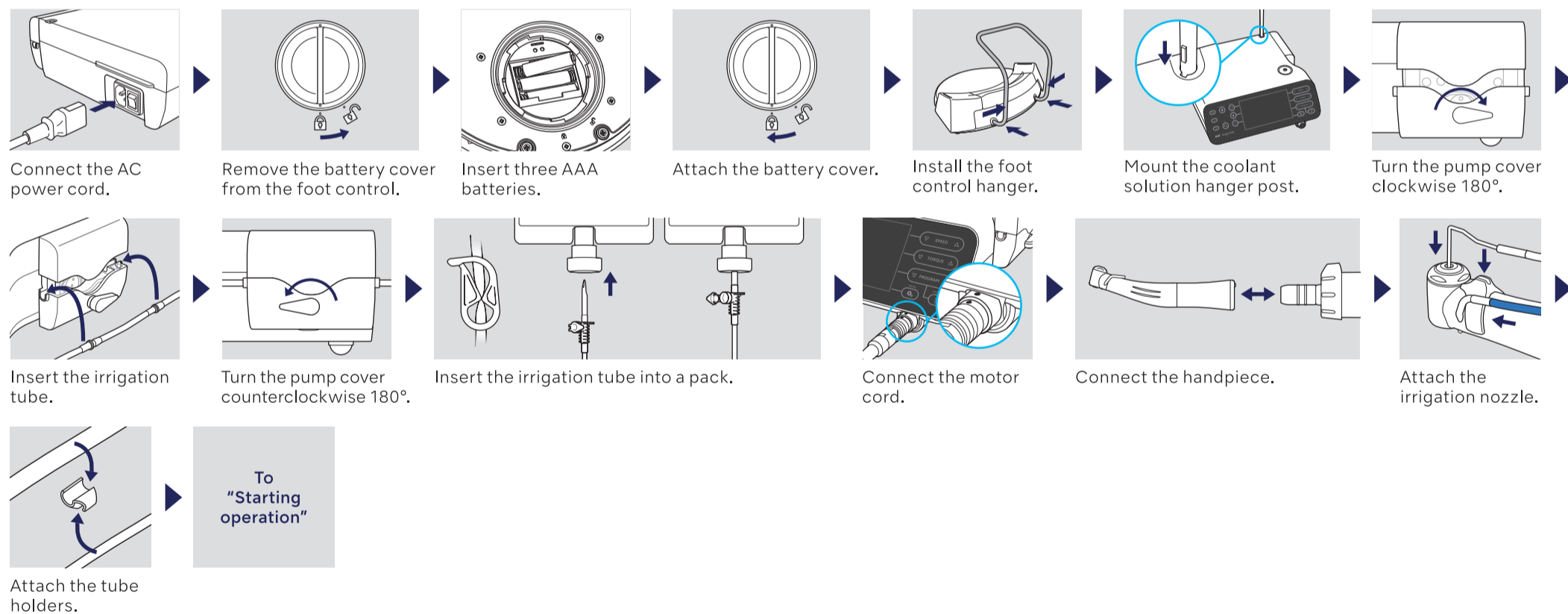

## Starting operation

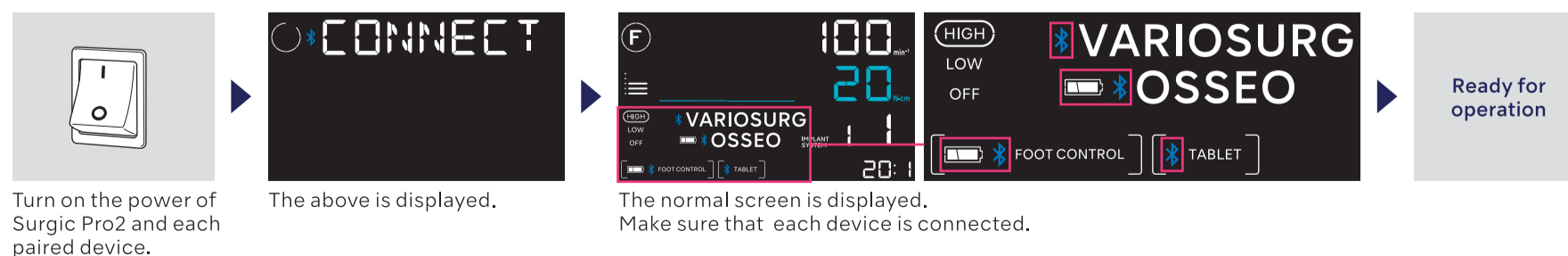

# The first step after purchase (Pairing with each device)

**1**  
At least 2 sec.  
Press and hold the SETTING Key. "BLE SET" is displayed.

**2**  
Press the ENTER Key.


**3**  
Press the TORQUE Key to select the device. NOTICE : The letters on the screen except "BLE SET" and "PAIRING" flash on and off.





**Foot control**  
PAIRING FC

**Osseo 100+**  
PAIRING OS

**VarioSurg3**  
PAIRING LK


**External terminal**  
PAIRING PC

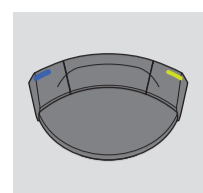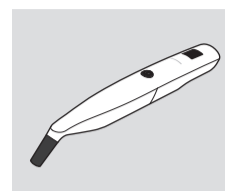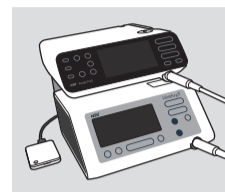

Press the ENTER Key.

## ▼ Foot control

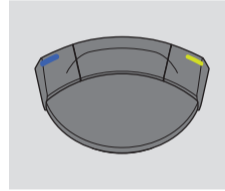

This operation is not necessary when using Surgic Pro2 for the first time after purchase. If the foot control is unpaired or when using another foot control newly purchased, conduct the following operation. NOTICE : Paring can be done within 10 minutes after inserting the battery.

Press the ENTER Key.


Approx. 10 sec.  
Press and hold the Left and Right buttons and the pedal at the same time.

PAIRING OK

Connected  
FOOT CONTROL

Press the CANCEL Key to return to the normal screen.

## ▼ Osseo 100+

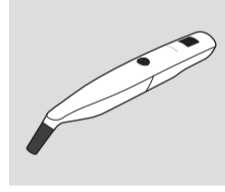

Press the operating key to turn on the power.

At least 3 sec.  
Press and hold the operating key.


"PA" is displayed.

PAIRING OK

Connected  
OSSEO

Press the CANCEL Key to return to the normal screen.

## ▼ VarioSurg3

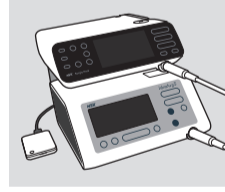

Turn on the power.

Connect the link cable beforehand.

Press and hold the button.


When the LED turns blue, release the button.

PAIRING OK

Connected  
VARIOSURG

Press the CANCEL Key to return to the normal screen.

## ▼ External terminal

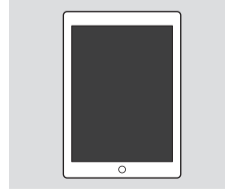

Launch the App.

Install the App beforehand.

Touch the icon.

Touch "Start".

Bluetooth Pairing Request  
"SurgicPro2" would like to pair with your iPad. Enter the code shown on "SurgicPro2".

Enter the code "123456" and touch "Pair".

PAIRING OK

Connected  
TABLET

Press the CANCEL Key to return to the normal screen.



[www.nsk-dental.com/qr/app-surgicpro2](http://www.nsk-dental.com/qr/app-surgicpro2)